

Nuview Loupe and Light Visor

The Nuview Loupe and Light Visor system has been designed for use with frame mounted loupe/light combinations. The extended nature of the visor area, whilst complying to the requirements of EN166 2001, provides sufficient space to accommodate most types of loupes including prismatic loupes with a "front" mounted light.

The Nuview Loupe and Light Visor system has the following features

- Headband mounted visor avoiding additional weight on the frames
- Superior optical quality visor ensures clarity
- Adjustable foam padded headband for maximum comfort
- Easily detached visor mount for cleaning and disinfecting
- Daily disposable visors avoiding the poor image quality associated with re-usable visors.
- Competitively priced multipacked visors
- Tested effective to EN 166:2001 and CE marked accordingly.

The visors are designed to be replaced daily after use and are supplied in packs of 10. Should the visor become scratched, discard the visor and replace with a new one. If cleaning and disinfecting the visor this should be undertaken in compliance with the



practice/institution approved infection control procedures.

The standoff can be removed for cleaning and disinfecting in compliance with the practice/institution approved infection control procedures.

For headband storage with a visor attached invert the headband on a surface when not in use.

The Nuview Loupe and Light Visor system has been tested effective to EN 166:2001 and is CE marked accordingly.

Nuview Ltd, Vine House, Selsley Rd, North Woodchester, Glos GL5 5NN

T +44(0)1453 872266

F +44(0)1453 872288

E info@nuview-ltd.com

W www.nuview.co



Nuview Loupe and Light Visor

Instructions for Use

Fitting the Visor

1. Starting from one side, attach the standoff to the Velcro strip on the front of the headband, taking care to ensure it is central on the headband.
2. Apply pressure to the standoff to ensure adhesion of the Velcro.



3. Remove the visor from the packaging.
4. Press the visor onto the oblong standoff tabs as shown, starting at one side and working around the standoff, making sure the visor sits tight and flat against it.



5. Adjust the headband for a comfortable fit by twisting the rear thumbwheel and pressing the peg into the appropriate hole on the top band.



Re-ordering details

Pack of 1 Headband

PPE-HB-BLACK

Pack of 2 Headband Standoffs

PPE-HB-30001

Pack of 10 Headband Visors

PPE-HB-2022

Nuview Ltd, Vine House, Selsley Rd, North Woodchester, Glos GL5 5NN

T +44(0)1453 872266

F +44(0)1453 872288

E info@nuview-ltd.com

W www.nuview.co

